

Duodenal Mucosal Resurfacing Durably Maintains Weight Loss in Metabolic Disease

Kelly White, Emily Cozzi, Joseph Giaconia, Hui Zhang, Timothy Kieffer, Jay Caplan, Harith Rajagopalan

Fractyl Health, Inc., Burlington, MA, USA




Introduction

- Drugs for obesity are effective for weight loss; however, discontinuation is frequent and results in weight rebound, underscoring the need for weight-maintenance therapies.
- The duodenal mucosa plays a key role in metabolic regulation and is known to be impaired in metabolic disease (Figure 1).¹⁻¹¹
- Duodenal mucosal resurfacing (DMR) is an investigational, non-drug, minimally invasive, endoscopic procedure that uses hydrothermal ablation to restore duodenal metabolic function (Figure 1).¹²⁻¹³
- Clinical trials with >300 patients have shown that DMR may safely improve multiple indices of metabolic health including glycemic control, insulin sensitivity, hepatic fat, and weight while reducing medication burden.¹³⁻¹⁹

The current pooled analysis was undertaken to evaluate the durability of DMR-induced, weight-related outcomes.

Figure 1. Rationale for Targeting Duodenal Dysfunction with DMR.

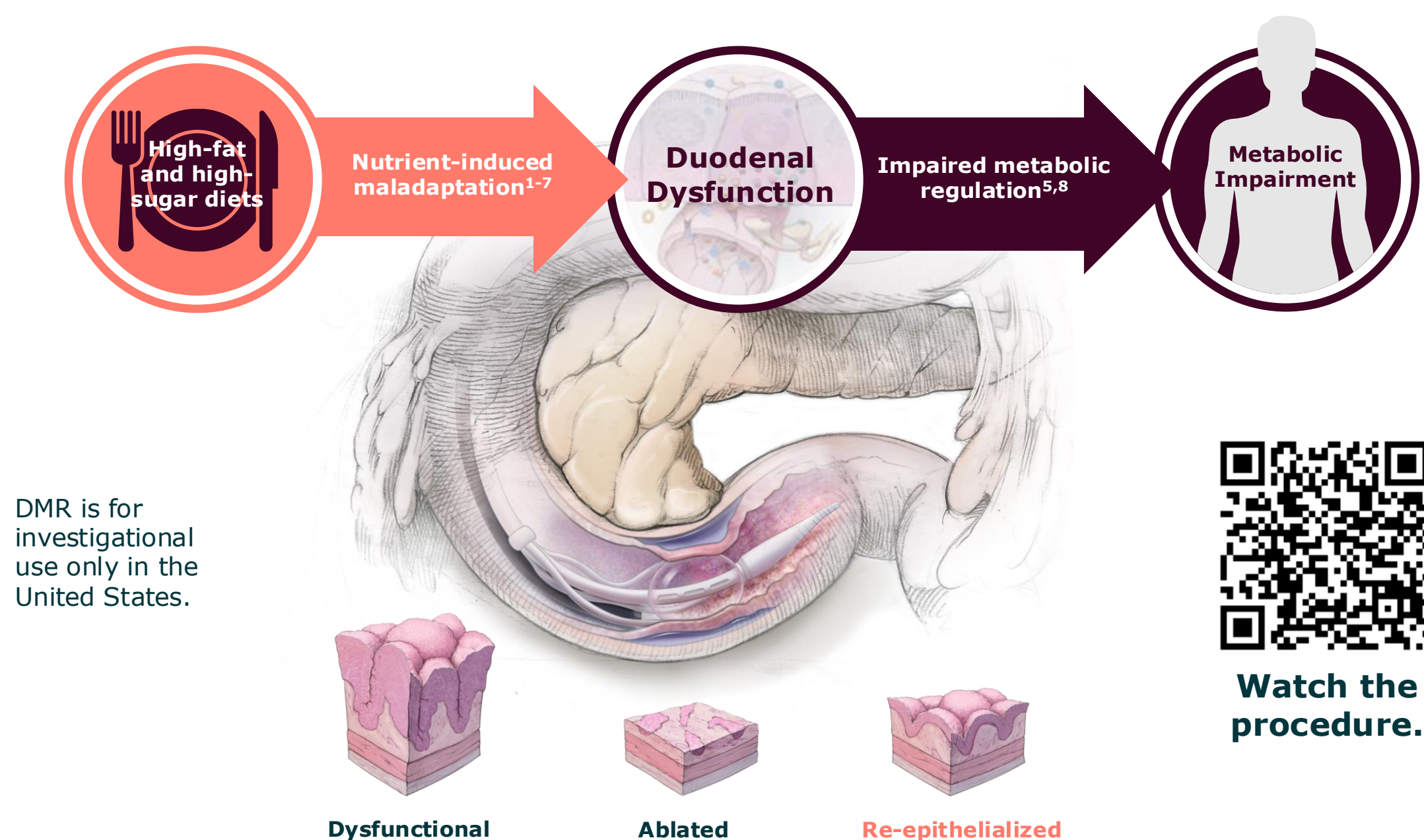

DMR is for investigational use only in the United States.

Study Design

- Included pooled data from 5 clinical trials from N=118 participants in Europe or the United States followed for 48 weeks post-procedure.
- Trials were conducted from 2015 to 2023.
- No diet or lifestyle intervention changes were made after the DMR procedure.
- Participants in which an obsolete version of the DMR catheter (double catheter) was used were excluded.

Results

Table 1. Demographics and Baseline Characteristics. Participants had longstanding, inadequately controlled type 2 diabetes. Most had obesity (62%) or overweight (34%) at baseline.

Demographics	N=118
Male, n (%)	88 (75)
Age (years), mean (SD)	58 (8)
Baseline Characteristics	N=118
Diabetes duration (years), mean (SD)	10 (5)
HbA1c (%), mean (SD)	8.2 (0.7)
Body weight (kg), mean (SD)	93 (14)
BMI (kg/m ²), mean (SD)	31.1 (3.8)
With obesity (BMI>30 kg/m ²), n (%)	73 (62)
With overweight (BMI>25 and ≤30 kg/m ²), n (%)	40 (34)

Figure 2. DMR Led to Sustained Weight Loss in the Majority of Participants. Individual changes in body weight from baseline at 48 weeks post-DMR are shown (n=94).

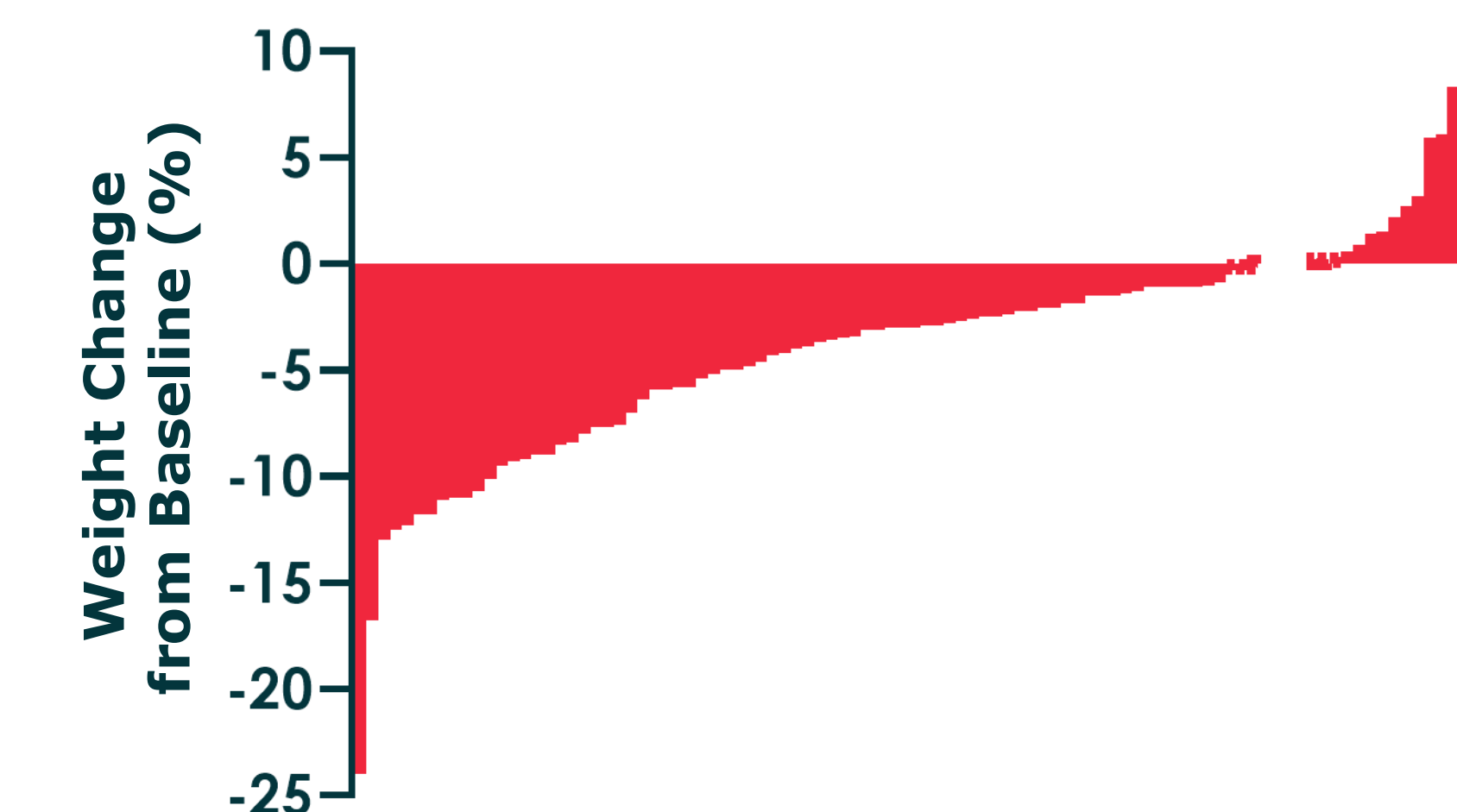


Figure 3. DMR Durably Maintained Weight Loss Through 48 Weeks Post-Procedure. DMR induced a mean (SEM) weight loss of 3.4% (0.3%) at 4 weeks, 3.7% (0.4%) at 12 weeks, 3.9% (0.4%) at 24 weeks, 3.7% (0.4%) at 36 weeks, and 4.0% (0.5%) at 48 weeks (all p<0.0001 vs. baseline). Weight maintenance also was observed through 48 weeks after censoring participants who added any glucose-lowering agent that may have contributed to weight loss (e.g., GLP-1RA or SGLT2i, data not shown) during follow-up. Weight change from baseline was evaluated by paired t-test. Data are shown as mean ± SEM.

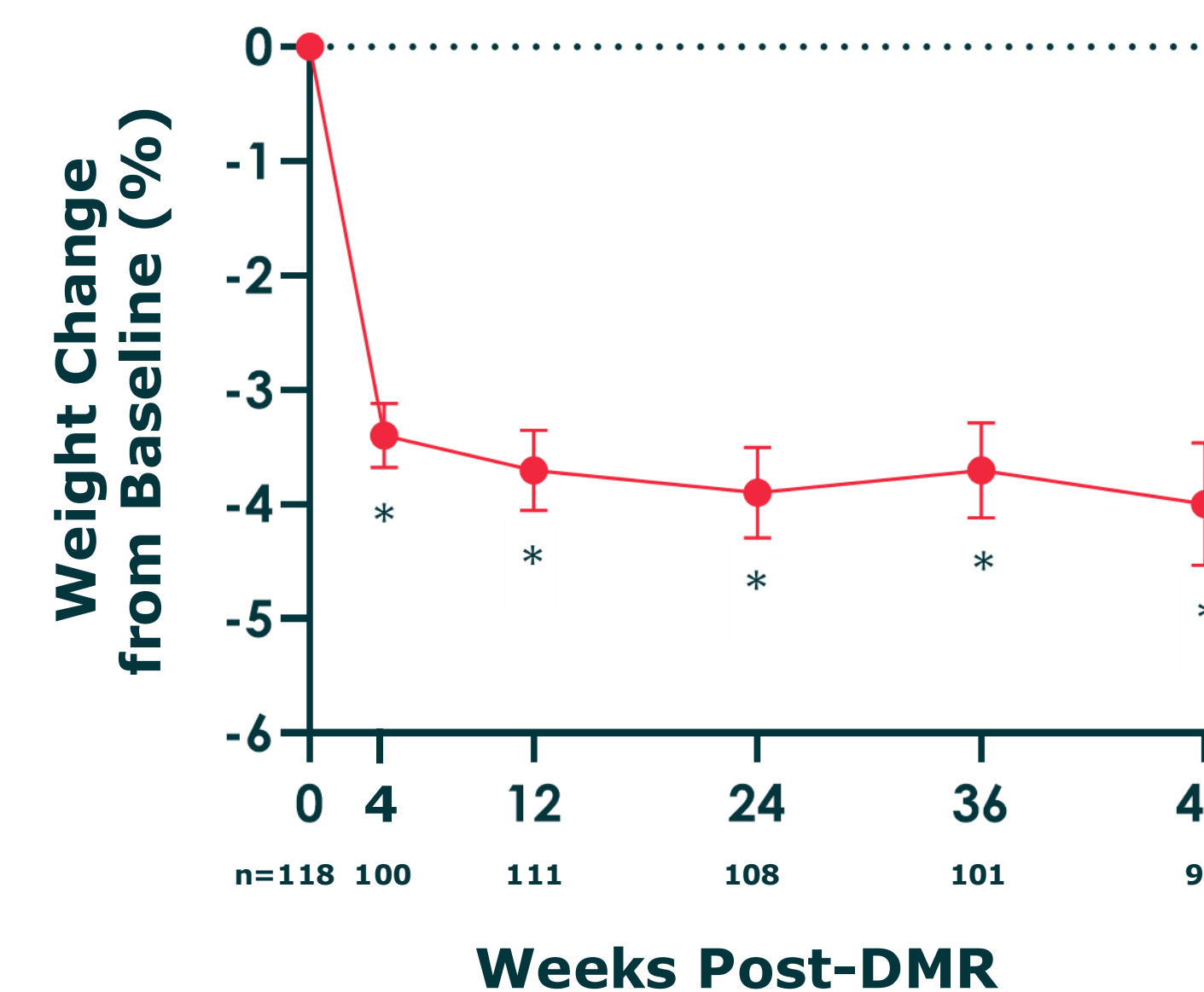


Table 2. The Majority of Participants Who Lost Weight at 4 Weeks Maintained Their Weight at 48 Weeks Post-Procedure. Of the participants who had weight data at both week-4 and -48 post-procedure visits (n=78), 90% achieved weight loss at week 4. Of these (n=70), 84% maintained their weight loss at week 48, and weight was stable from week 4 to week 48 in the cohort.

Category	n=78
Lost weight at 4 weeks post-DMR, % (n)	90 (70 of 78)
Maintained 4-week weight loss at week 48, % (n)	84 (59 of 70)
Mean change in weight from weeks 4 to 48, % (SEM)	0.2 (0.5)

Table 3. Overall Safety Summary. The DMR procedure was well tolerated. No serious device- or procedure-related AEs were observed.

Device/Procedure-Related AEs,** n (%)	Participants with ≥1 Event (n=117*)
Most Common Device/Procedure-Related AEs, n (%)	
Abdominal pain	20 (17)
Oropharyngeal pain	12 (10)
Nausea	6 (5)
Diarrhea	5 (4)
Abdominal pain upper	3 (3)
Vomiting	3 (3)
Device/Procedure-Related Serious AEs, n	0

*N=118 in pooled ITT population; n=1 randomized to DMR but did not receive treatment.
**Device/Procedure Related AEs include definitely or probably related to procedure or device.

Conclusion and Next Steps

These data demonstrate that a single DMR procedure may safely result in durable weight maintenance through 48 weeks in patients with type 2 diabetes.

The impact of DMR on weight maintenance in patients with obesity, who discontinue GLP-1 therapy, will be assessed in the currently enrolling REMAIN-1 trial.

Publications and Presentations



Remain1study.com



Clinicaltrials.gov



Abbreviations: AE=adverse event, BMI=body mass index, DMR=duodenal mucosal resurfacing, GLP-1RA=glucagon-like peptide 1 receptor agonist, ITT=intent to treat, SD=standard deviation, SEM=standard error of the mean, SGLT2i=sodium-glucose cotransporter-2 inhibitor

References: 1. Mah AT et al. Endocrinology. 2014;155:3302-3314. 2. Baldassano S et al. J Endocrinol. 2013;217:11-20. 3. Mao J et al. Diabetes. 2013;62:3736-3746. 4. Alluave A et al. Nat Metab. 2021;3:1202-1216. 5. Dailey MJ. Physiol Behav. 2014;136:74-78. 6. Theodorakis MJ et al. Am J Physiol Endocrinol Metab. 2006;290:E550-559. 7. Verdam FJ et al. J Clin Endocrinol Metab. 2011;96:E379-E383. 8. Ghilii et al. Diabetologia. 2010;53:2233-40. 9. Fiorentino et al. Obesity (Silver Spring). 2023;31:724731. 10. Dyer J et al. Am J Physiol Gastrointest Liver Physiol. 2002;282:G241-G248. 11. Fiorentino et al. J Clin Endocrinol Metab. 2017;102:3979-3989. 12. de Moura EGH et al. Endosc Int Open. 2019;7:E685-E690. 13. Haldry RJ, et al. Gastrointest Endosc. 2019;90:673-681.e2. 14. van Baar ACG, et al. Endosc Int Open. 2020;8:E1683-E1689. 15. Rajagopalan H, et al. Diabetes Care. 2016;39(12):2254-2261. 16. van Baar ACG, et al. Gut. 2020;69:295-303. 17. Mingrone G, et al. Gut. 2022;71:254-264. 18. van Baar ACG, et al. Gastrointest Endosc. 2021;94:111-120.e3. 19. van Baar et al. Diabetes Res. Clin. Pract. 2022;184:109194.

Disclosures: All authors are employees and shareholders of Fractyl Health, Inc. Harith Rajagopalan is a Board member of Fractyl Health, Inc.

Acknowledgments: Funded by Fractyl Health, Inc. The authors thank Edward Mahoney for writing and editorial support (Fractyl Health, Inc.) and Second Melody for design and creative support.

Contact information: To learn more about the presented research and/or Fractyl Health, Inc., please visit our website at <https://www.fractyl.com> or email@fractylhealth.com.

